

Dux Dental
 Zonnebaan 14, NL-3542 EC Utrecht
 The Netherlands
 Tel. +(31) 30 241 0924
 Fax. +(31) 30 241 0054
 Email: info@dux-dental.com
 www.duxdental.com

Trust.Worthy.Innovation.



IMAGE
 Alginate



The perfect mix with the Alginator II

The Alginator II mixing machine makes a smooth, bubble-free mix in a jiffy, while eliminating waste and reducing mixing and clean-up time significantly.

Image Alginate canister	500 g + canister	Fast set (2 Minutes)	Regular set (3 Minutes)
Image Alginate Value pack	10x 500 g + canister	Ref. 27416	Ref. 27426
Image Alginate Refill pack	11x 500 g	Ref. 27417	Ref. 27427
Image Alginate Institutional pack	50x 500 g	Ref. 27419	Ref. 27429
Image Alginate Singles	52 units/box	Ref. 27418	Ref. 27428
Image Alginator II	Mixing machine incl. 2,5 kg Alginaat	Ref. 27412	Ref. 27412
Image Syringable impression-taking system	48 cartriloids, 1,8 ml (Syringe material blue)	Ref. 25233	Ref. 25232
Image Syringable impression-taking system	Starter Kit	Ref. 21600	Ref. 21601
Image Syringable impression-taking system	Starter Kit incl. DryProcessor	Ref. 25300	



For more information or a free Image Alginate trial package, please call us:
 00 31 30 241 0924 or send us an E-mail: info@dux-dental.com !

Is it a silicone? No, it's Image™ Alginate!



Image Alginate is dust free, very smooth, creamy and easy to mix.

The green color makes all details of your impression easily visible. Image Alginate has a fresh mint flavor and is a natural, 100% bio-degradable product.

*Image Alginate.
 Unsurpassed smoothness
 and amazing detail.*

Image Alginate product features and benefits

Water-loving formula

Uniform particle size assures a precise rate of absorption. All ingredients are soluble at the same rate yielding a quick, uniform and smooth mix.

Unsurpassed surface smoothness and green color

Your impressions will have a glass-like finish and the green color makes them easy to read. You will see the difference in your stone models and in your provisional restorations.

Dust-free

The alginate will not dust during dosing and mixing.

Friendly to the environment

100% bio-degradable.

Fresh mint flavor

Image Syringable

Image Syringable is a cost saving, quick and precise two component alginate-hydrocolloid impression-taking system, ideal for single crowns, three unit bridges, removable partial dentures, inlays and onlays.



2

Application. Dispense Image Syringable rapidly and in bulk directly onto the marginal areas and contours of the moistened prepared teeth.



3

Impression-taking. Seat the tray with Image Alginate over the fluid syringe material.



1

Preparation. Image Syringable is boiled and kept in the DryProcessor and after processing ready for use during the whole week.



4

Tray removal. Hold until the alginate is completely set and remove the tray with a quick snap.



5

Rinse. Rinse the impression for about 30 seconds under running water.



6

Disinfect. Disinfect the impression using your preferred disinfectant, following the manufacturer's instructions.

7

Pour. Rinse the impression with tap water once more and pour immediately for best accuracy.

Image Alginate Singles

The convenient, efficient and hygienic alginate solution for *high quality* alginate impressions. Image Alginate is now also available in single impression units: "Image Alginate Singles".

Quality

Always the right measure of alginate and water. The mixture viscosity will be exactly the same every time.

Efficient

No need to use a scoop and vial to measure alginate and water and no more space consuming alginate canister or bag.

Hygienic

No more unwashed water vial gathering bacteria. A personal alginate packaging for every patient.



actual size

New!
Image Singles
single impression
units